

Empulse R90<sup>®</sup>

Top **tips**  
to maximise  
the **R90's**  
**battery** life

POWER ASSIST  
DEVICE



# 1. Click out the battery overnight to preserve its charge and optimal performance.

Slightly lifting the battery is enough to disconnect it.





This button controls the **Wheel Up/Down** function—not power or battery disconnection.





**2. Charge the battery after every use** as a recommended practice.


### **CHARGING TIME**

Standard (2.5 Ah):  $\approx$  1 hour  
Extended (5 Ah):  $\approx$  2 hours



### 3. Use the **Sunrise Intelligence App** to adjust the **settings** to suit your typical usage.

High-performance settings can increase battery consumption.

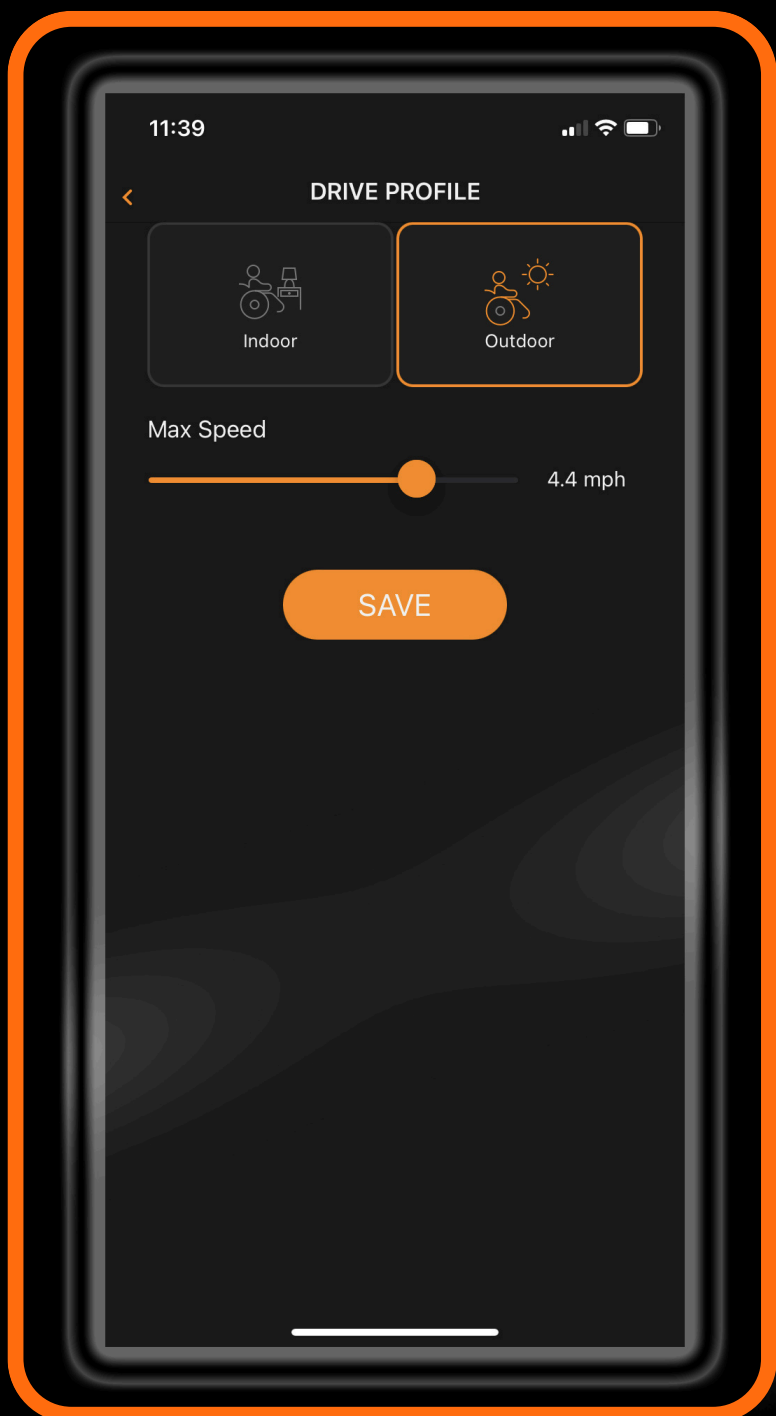

**4. Store** the battery in a **cool, dry place**, ideally between 10°C (50°F) and 25°C (77°F).



**5.** Like all lithium-ion batteries, keep the **charge** at approximately **75%** when **in storage**.



Low charge for extended periods risks deep discharge and permanent damage. To prevent this, **recharge every 12 weeks during storage.**

