




Please read these instructions carefully before beginning the assembly. Failure to understand and follow assembly instructions may result in injury to technician and/or end user and may void the warranty. If you have any questions regarding the R90 call Sunrise Medical Technical Support at +1 (800) 333-4000.

Sunrise Medical is not responsible for the compatibility and alignment with competitor chairs and is not obligated to update compatibility information for competitor products. This guide provides recommendations for the required mounting hardware for these specific wheelchairs, but it is essential to evaluate the chair's configuration to ensure full compatibility with the R90. This instruction was created with the competitor chair information available at the time of publication.

Motion Composite Wheelchairs:

Helio A7, Helio C2, Helio, Veloce, and KIDS

R90 Folding Adapter Size: Medium

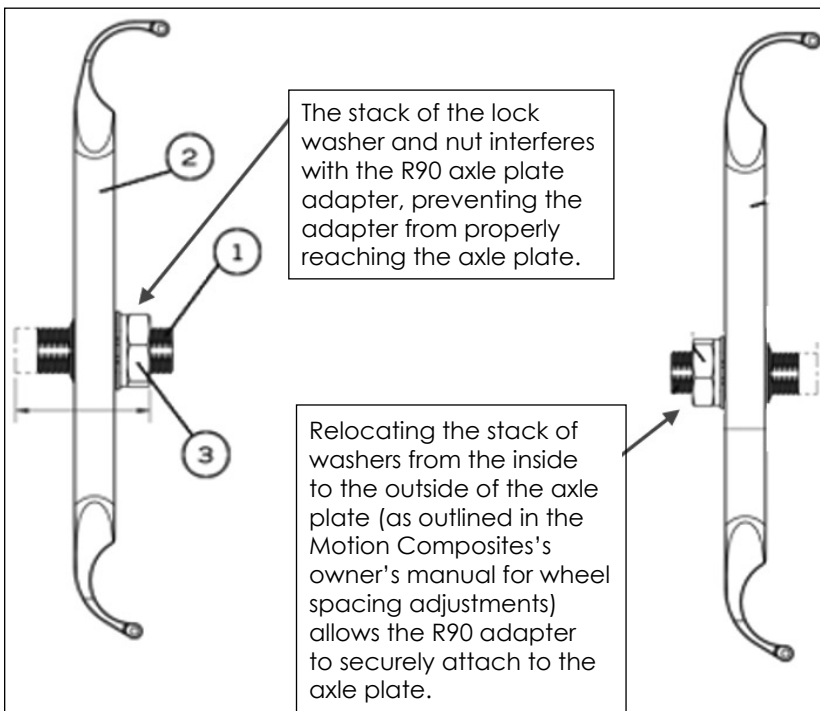

Section 8.11.5 of the Motion Composite Folding Wheelchair User Manual provides direction on how to move the washer and nut stack from the inside to the outside of the axle plate.

Source: 0070757E-00 MC Folding Wheelchair Manual, Revision Date 2025-08-06. Access user manual at: www.motioncomposites.com

8.11.5 ADJUSTING REAR WHEEL SPACING

- The rear wheels can be adjusted laterally by repositioning axle receiver (1) on mounting plates (2).
- Loosen nuts (3) on axle receivers (1).
- Position nuts (3) to adjust the spacing.
- Tighten the nuts (3) firmly.
- The nut can be switched from side to side for more adjustments.



DEALER/TECHNICIAN WARNING

Attention dealers and qualified technicians, do not operate or service the R90 without first reading the owners manual. If you do not understand the instructions and warnings in the R90 owner's manual please contact the Sunrise Medical Technical Service Department before operating and/or servicing the R90 device. Failure to do so may result in damage and/or injury.

Find more information and important warnings in the device owner's manual or at: www.SunriseMedical.com.