

QUICK START GUIDE

CTRL+5



**"HELLO,
SUNRISE"**

This guide will help you get started with your CTRL+5. For detailed instructions, installation information, or local contact details, please refer to your owner's manual.



SWITCH-IT®

SUNRISE MEDICAL CARES ABOUT YOUR SAFETY.
Please read your owner's manual for safe operating
instructions and maintenance guidelines.

HOW IT WORKS

The **CTRL+5** lets you operate your power seating functions using intuitive **push buttons**, **toggle switches**, or **voice commands**. Voice activation is available on both switch box versions. However, you can always operate CTRL+5 manually using the buttons, even if voice activation is turned on.

Buttons 1-10 come pre-programmed with default seating functions but can be re-programmed by a dealer.

CTRL+5 gives you seamless access to your preferred positions — helping you maintain optimal posture throughout the day.

USING CTRL+5 WITH BUTTONS OR TOGGLE SWITCHES

- **Power on** your wheelchair.
- **Press and hold the button or toggle** (depending on model) to move the seat
- **Release to stop** immediately.



- Enter the settings mode by holding buttons **1, 9, and 10 for 3 seconds**
- To turn on voice activation, press **button 7**

FIRST SET UP CTRL+5 WITH VOICE CONTROL

- **Voice activation is OFF by default.**
- **Power on** your wheelchair.
- **To turn it on**, enter the settings mode by holding buttons **1, 9, and 10 for 3 seconds** → LED will turn **RED** alerting you are in the settings mode.
- To turn on voice activation, press **button 7**.
- **Hold buttons 1 + 9 + 10 again** to save and return to normal mode

USING CTRL+5 WITH VOICE CONTROL

- **Power on** your wheelchair.
- Say **“Hello Sunrise”** → LED will turn **BLUE**.
- Say **“Sunrise [number]”** (e.g., “Sunrise 2”) to activate a power seating function for a fixed period of time that you choose.
- **Listen** for next command for **8 sec.**

TURN VOICE CONTROL ON/OFF & SET ACTIVATION TIME

- **Hold buttons 1, 9, and 10 for 3 seconds**
→ LED will turn **RED** alerting you are in the settings mode.
- Press **Button 7** to **increase** activation time
- Press **Button 8** to **decrease** activation time
- LED will turn **PURPLE** when increasing/decreasing activation time
- **Hold buttons 1 + 9 + 10 again** to save and return to normal mode

Each LED will indicate 6 seconds of activation time:

- No LEDs = Voice control OFF
- 1 LED = 6 sec
- 2 LEDs = 12 sec
- 3 LEDs = 18 sec
- 4 LEDs = 24 sec
- 5 LEDs = 30 sec

CHANGE VOLUME LEVEL

- **Hold buttons 1, 9, and 10 for 3 seconds**
→ LED will turn **RED** alerting you are in the settings mode.
- Press **Button 3** to **increase** volume
- Press **Button 4** to **decrease** volume
- LED will turn **CYAN** when increasing/decreasing activation time
- **Hold buttons 1 + 9 + 10 again** to save and return to normal mode

Each LED will indicate 20 %:

- No LEDs = Mute
- 1 LED = 20%
- 2 LEDs = 40%
- 3 LEDs = 60%
- 4 LEDs = 80%
- 5 LEDs = 100%



← LED Lights

- Enter the settings mode by holding buttons **1, 9, and 10 for 3 seconds**
- Press **Button 7** to **increase** activation time
Press **Button 8** to **decrease** activation time
- Press **Button 3** to **increase** volume
Press **Button 4** to **decrease** volume

LED COLOR GUIDE

- **Blue:** Listening (after “Hello Sunrise”)
- **Green:** Command active – system is ready
- **Yellow:** Command active – drive inhibit is active
- **Red:** Signals a lockout drive inhibit
- **Purple:** Adjusting activation time
- **Cyan:** Adjusting volume level

Scan the QR Code to view this document in other languages and a “how to” video.



toggle switches



push-button



Sunrise Medical (US), LLC
©07.2025



www.SunriseMedical.com