



JAY Your Way is a modification program designed to deliver the benefits of personalized seating but with the ease of ordering standard JAY products.





From clinical improvements to colored covers, JAY Your Way modifies cushions and backs specifically to address users' unique individual needs and preferences. This brochure includes information on commonly used modifications for JAY bases, fluid inserts, covers, backrests and JAY products. Simple instructions to ordering JAY Your Way are also included and most modifications are made at no additional charge. With JAY Your Way, JAY products are easily modified to meet your clients' needs and individual styles.

TABLE OF CONTENTS:

CUSHION MODIFICATIONS

| | |
|--|-----|
| Size Modifications | 3 |
| Base and Positioning Modifications | 4-5 |
| Positioning Components | 6 |
| Fluid Insert Modifications | 7 |
| Cover Modifications | 7-8 |

BACKREST MODIFICATIONS

| | |
|--------------------------------|---|
| JAY J3 Back Cover Colors | 8 |
| JAY J3 Back Shell Colors | 9 |
| Military Service Patches | 9 |

ARM TROUGH AND FOOT BOX MODIFICATIONS

| | |
|------------------------------------|-------|
| CUSHIONS MODIFICATION MATRIX | 11-13 |
|------------------------------------|-------|

| | |
|---------------------------------|----|
| BACKS MODIFICATION MATRIX | 14 |
|---------------------------------|----|

| | |
|--|----|
| ARM TROUGH AND FOOT BOX MODIFICATIONS MATRIX | 15 |
|--|----|

| | |
|--------------------|------------|
| HOW TO ORDER | Back Cover |
|--------------------|------------|



Cushion Base Modifications

Size Modifications



Reduce Size

Modification Detail: Remove foam from the length or width of the cushion base and sew the cover to fit

Clinical Application: Maintain the size of the Pelvic Loading Area (PLA) but decrease the size of the cushion



Increase Size

Modification Detail: Add equivalent foam or cut down a larger base to the length or width of the cushion base and sew the cover to fit

Clinical Application: Maintain the size of the PLA and add foam to increase lateral stability



Reduce Cushion Height

Modification Detail: Remove foam from the cushion base to decrease height

Clinical Application: Accommodate wheelchair fit, environment needs, and/or client needs

Example: Decrease seat-to-floor height for easier transfers and foot propulsion




Increase Cushion Height

Modification Detail: Add foam to cushion base to increase height

Clinical Application: Accommodate wheelchair fit, environment needs, and/or client needs

Example: Increase seat to floor height for easier transfers and foot propulsion

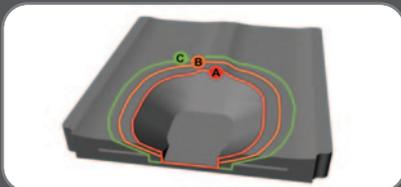

Well Sizing

Modification Detail: Increase or decrease the standard, factory set size of the PLA

Clinical Application: Allow for anatomically optimized loading areas when considering proper immersion, pressure reduction, and lateral stability



Move Pelvic Loading Area (PLA)

Modification Detail: Move the PLA forward towards the front of the cushion

Clinical Application: Accommodate kyphotic postures or clients who require increased room between PLA and backrest


Base and Positioning Modifications



Notch for Seat Rails

Modification Detail: Remove foam along the bottom, lateral edges of the foam base

Clinical Application: To lower the cushion between seat rails to achieve the lowest possible seat-to-floor height

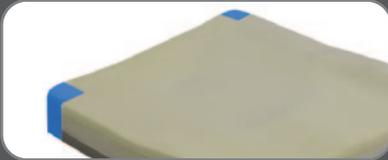

Notch for Back Posts

Modification Detail: Remove foam from the back corners of the foam base

Clinical Application: Allows cushion to slide between back posts when loading surface is posterior to back posts, to lock in cushion in the case of frequent transfers, and to build in additional depth growth opportunities



Leg Length Cut (Right/Left)

Modification Detail: Decrease the length of the foam base and fluid pad (if applicable) on the left or the right side and sew the cover to fit


Clinical Application: To accommodate varying femoral lengths



Contracture Cut

Modification Detail: Undercut the front of the foam base at an angle


Clinical Application: To clear the calves in cases of tight hamstrings, large calves, or close front thigh to lower leg angles, to accommodate chair configurations, and to accommodate foot propulsion

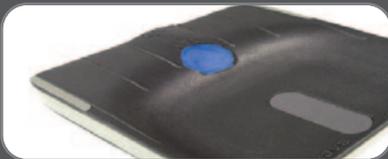

Scrotal Relief Cut

Modification Detail: Scoop out the front and center of the pelvic loading area (PLA)

Clinical Application: To accommodate organ tissue, reduce discomfort, and increase sitting tolerance

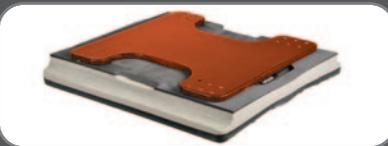

Personalized Drop Seat

Modification Detail: Fit a drop seat to the cushion base

Clinical Application: To minimize seat-to-floor height for foot propulsion and transfers



Angle Cut Posterior Slope

Modification Detail: Remove foam at an angle from underneath the rear of the cushion base; front of cushion will be higher than the back by specified dimension

Clinical Application: To close seat to back angle (e.g. for postural stability)

Base and Positioning Modifications



Angle Cut Anterior Slope

Modification Detail: Remove foam at an angle from underneath the front of the cushion base; back of cushion will be higher than the front by specified dimension

Clinical Application: To accommodate hip flexion limitations and assist with foot propelling and transfers



Wedge Glue Posterior Slope

Modification Detail: Add a foam wedge underneath the front of the cushion base; front of cushion will be higher than the back by specified dimension

Clinical Application: To close seat to back angle (e.g. for postural stability)



Wedge Glue Anterior Slope

Modification Detail: Add a foam wedge underneath the front of the cushion base; back of cushion will be higher than the front by specified dimension

Clinical Application: To accommodate hip flexion limitations and assist with foot propelling and transfers



Medial Support Cut

Modification Detail: Remove the pre-contoured medial support

Clinical Application: To improve positioning when femoral alignment is not neutral (e.g. in windswept postures) or when transfers are inhibited by the medial thigh support



Femoral Relief Cut

Modification Detail: Shave away the front of pelvic loading area (PLA) to transition from PLA to femoral support area


Clinical Application: To reduce pressure at front of PLA area to accommodate tissue and increase sitting tolerance for users with sensation



Leg Trough Cut

Modification Detail: Scoop 1" of base material from the front of the PLA forward to the front edge of the femoral support area

Clinical Application: To improve medial and lateral positioning and support of the femurs and to accommodate hip limitations by opening hip angles

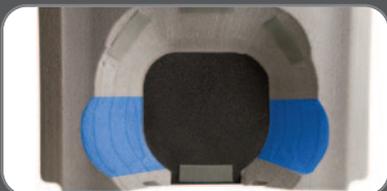

Trochanteric Relief Cut

Modification Detail: Remove material at the lateral borders of the PLA, beneath the trochanters

Clinical Application: To reduce pressure at, or surrounding, the trochanters and evenly distribute load across the whole sitting surface


Positioning Components



Fixed Lateral Thigh Positioning Pieces

Modification Detail: Add positioning pieces to lateral edge of the cushion base


Clinical Application: To improve sitting posture and draw legs toward median of the body

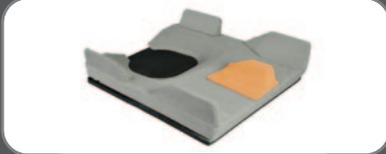

Fixed Medial Thigh Positioning Piece

Modification Detail: Add positioning pieces to the middle of the cushion base

Clinical Application: To improve sitting posture and position inner thighs

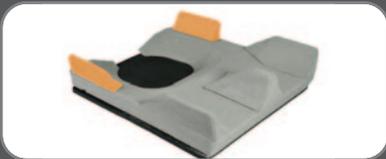

Fixed Lateral Pelvic Positioning Pieces

Modification Detail: Add soft foam positioning pieces to the inner cover at rear of the cushion

Clinical Application: To improve sitting posture and position pelvic area



Removable Lateral Thigh Positioning Pieces

Modification Detail: Add soft foam positioning pieces to the inner cover at lateral edge of the cushion

Clinical Application: To improve sitting posture and draw legs toward median of the body



Removable Medial Thigh Positioning Pieces

Modification Detail: Add soft foam positioning pieces to the inner cover in the front and middle of the cushion


Clinical Application: To improve sitting posture and position inner thighs



Removable Lateral Pelvic Positioning Pieces

Modification Detail: Add positioning pieces to the rear of the cushion base

Clinical Application: To improve sitting posture and position pelvic area

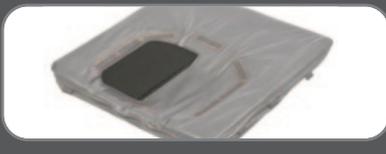

Obliquity Pads

Modification Detail: Removable foam pads placed under the JAY fluid pad on either right or left side of the pelvic loading area (PLA)

Clinical Application: To accommodate bone or tissue asymmetry on one side of the client in the case of a flexible obliquity



Visco Foam Overlay

Modification Detail: Soft Visco foam is added to the top of the cushion for added comfort

Fluid Insert Modifications



Underfill

Modification Detail: Reduce factory set fluid volume by weight up to 20% in the JAY fluid pad (5% increments only)

Clinical Application: Increase immersion of the pelvis into the fluid to achieve appropriate skin protection and/or lateral stability



Overfill

Modification Detail: Increase factory set fluid volume by weight up to 40% in the JAY fluid pad (5% increments only)


Clinical Application: Redistribute load and reduce pressure on the bony prominences when a client experiences "bottoming out" or discomfort on a standard JAY fluid pad



Accommodate Obliquities

Modification Detail: Apply an overfill or underfill to either the right or the left side of the JAY fluid pad

Clinical Application: To optimize immersion (underfill), prevent bottoming out (overfill), or substitute for bone or tissue asymmetry on one side of the client in the case of a flexible obliquity

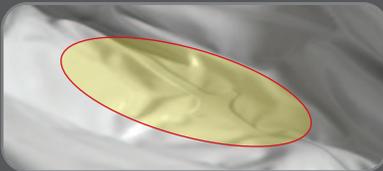

Sacral or Scrotal Seal

Modification Detail: Add a heat seal in the sacral or scrotal area of the JAY fluid pad

Clinical Application: To control the migration of fluid in order to reduce pressure in areas that are at risk for skin breakdown, pain, or discomfort

Cover Modifications

Each JAY cushion and back comes with a carefully designed cover manufactured from top-of-the-line materials that enhance their clinical effectiveness and comfort. JAY Your Way can be used to add some color or better fit users' unique lifestyles.



Zippered Inner Cover

Modification Detail: The Zippered Inner Cover is a stretchable and water-resistant cover used under the outer cover of the cushion. It is made from Reversed Dartex®, a water-proof, stretchy fabric, with seams stitched using anti-wicking thread and an AquaGuard® zipper.

Clinical Application: To reduce surface tension, to protect foam base from dirt and moisture build-up, and to prevent odor

Cover Modifications (Continued)

Personalized Outer Covers (Fabric Options)



Incontinent Zipper Cover



Incontinent Shower Cap Cover



Dartex® Cover with
AquaGuard® Zipper



Reverse Dartex® Cover
with Aquaguard Zipper



Microclimatic-3DX



JAY Stretch Zipper Cover



X-static® Cover with Breathable
Spacer Fabric



Embroidery

Modification Detail: Personalize your JAY cover with embroidery available in single color, Arial font up to 32 characters



Add Front Zipper Pocket

Modification Detail: Zippered pocket is sewn on the front underside of the cushion cover for convenient access and storage for small items such as keys, credit cards, or cell phones



Outer Cushion Cover with Colored Side Bands and Colored JAY Back Cover*



Digital Camo



Electric Lime



Grey



Hunters Orange



Navy Blue



Very Violet



Pink Sorbet



Shocking Pink



Turquoise



University Red



Volt Yellow

Backrest Modifications

Military Service Patches



Digital Camo back cover



Spinal Fluid Pad

Modification Detail: Add a JAY fluid pad over spinal area of foam back

Clinical Application: Reduce pressure along bony area of the spine that is at risk of skin breakdown



Visco Foam Insert

Modification Detail: Add a Visco memory foam insert with a cover sewn to match

Clinical Application: To add a comfort layer over the standard supportive foam for increased sitting tolerance and pressure reduction



Matte Purple painted back shell

Painted Back Shell (Colors)*

| | | | |
|--------------------|--------------|----------------------|-----------------|
| Aztec Gold | Bead Blast | Black Cherry | Black Opal |
| Blue Opal | Candy Blue | Candy Purple | Candy Red |
| Electric Blue | Evergreen | Glow | Green Apple |
| Hot Sparkle Pink | Mauve Pink | Rootbeer | Sparkle Silver |
| Sunrise Orange | Yellow | Titanium Color Paint | Matte Black |
| Matte Black Cherry | Matte Purple | Matte Electric Blue | Matte Evergreen |
| Glossy White | Black | | |

* The color options shown are for reference only. They are not intended as accurate representations. Refer to the Quickie® & Zippie® Color Brochure, Quickie & Zippie Color ring or the JAY color fabric sample for representations of painted colors. Print processes cannot accurately represent paint finishes or fabric textures. Actual colors may vary.

Arm Trough and Foot Box Modifications



Add Pommel to Hand Pad

Modification Detail: Add ergonomic pommel mount to hand portion of arm trough. Available for arm troughs only.

Clinical Application: To improve comfort and pressure distribution for improved sitting tolerance




Visco Overlay

Modification Detail: Add a top layer of Visco memory foam for additional comfort




Add Additional Strap

Modification Detail: Add additional strap to secure the extremity

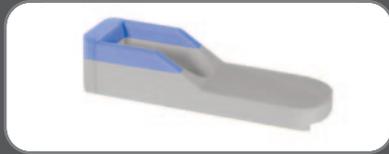


Reduce Height of Arm Troughs Side Wall

Modification Detail: Lower the height of one or both of the side walls. Available for arm troughs only.

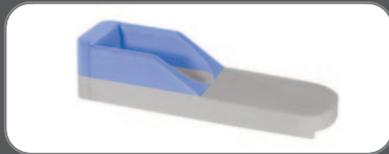

Remove Side Walls of Arm Trough

Modification Detail: Remove the side walls for larger extremities. Available for arm troughs only.

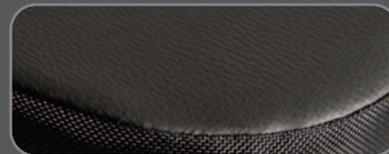

Naugahyde surface

Contact Surface

Fabrics: Naugahyde, Sheep Skin, and Microclimatic-3DX



Shocking Pink Side Bands

Colored Sidebands

Fabric Colors: See page 8 for color reference

JAY Your Way Cushions Modification Matrix

| | | Cushions | | | | | | | | | | | | | | | | | | |
|--|-----------------------------------|-----------------|-------------------|------------------------|----------------|------------------|------------------|--------------------|------------------|--------------------------|------------------|------------------|-------------------|-----------------|----------------------------|-------------------|---------------------|-----------------|----------------------------|-----------------|
| Modifications Offered | Additional Details | Upcharge Y or N | JAY Basic Cushion | JAY Basic PRO® Cushion | JAY GO Cushion | JAY Zip® Cushion | JAY Ion® Cushion | JAY Union® Cushion | JAY LITE Cushion | JAY Soft Combi P Cushion | JAY Care Cushion | JAY Duo® Cushion | JAY Easy® Cushion | JAY X2® Cushion | JAY J2®/J2 Recline Cushion | JAY J2 DC Cushion | JAY J2 Plus Cushion | JAY GS® Cushion | JAY Fusion® Series Cushion | JAY J3® Cushion |
| Size Modifications | | | | | | | | | | | | | | | | | | | | |
| Increase Length (added to front of cushion) | 1" | N | | | | | | | | • | | | | | • | • | | • | • | • |
| | 2" | N | | | | | | | | | | | | | | | | | | |
| | 3" | N | | | | | | | | | | | | | | | | | | |
| | 4" | N | | | | | | | | | | | | | | | | | | |
| Reduce Length (removed from front of cushion) | 1" | N | • | • | | | | | | • | | | | | • | • | | • | • | • |
| | 2" | N | • | • | | | | | | • | | | | | • | • | | • | • | • |
| | 3" | N | • | • | | | | | | • | | | | | • | • | | • | • | • |
| | 4" | N | • | • | | | | | | • | | | | | • | • | | • | • | • |
| Increase Width | 1" - 1/2" added to each side | N | | | | | | | | • | | | | | • | • | | • | • | • |
| | 2" - 1" added to each side | N | | | | | | | | | | | | | | | | | | |
| | 3" - 1 1/2" added to each side | N | | | | | | | | | | | | | | | | | | |
| | 4" - 2" added to each side | N | | | | | | | | | | | | | | | | | | |
| Reduce Width | 1" - 1/2" removed from each side | N | • | • | | | | | | • | • | | | | • | • | | • | • | • |
| | 2" - 1" removed from each side | N | • | • | | | | | | | | | | | | | | | | |
| Reduce Cushion Height | 1/2" | N | | | | • | | | | | | | | | | | | | | |
| | 5/8" | N | | | | | | | | | | | | | | | | | | |
| Increase Cushion Height | 1" | N | | | | | | | | | | | | | | | | | | |
| | 2" | N | | | | • | • | • | • | • | | | | | • | • | | • | • | • |
| | 3" | N | | | | • | • | • | • | • | | | | | • | • | | • | • | • |
| Increase PLA / Well Sizing | 1" - 1/2" increase to each side | N | | | | | | | | | | | | | | | | | | |
| | 2" - 1" increase to each side | N | | | | | | | | | | | | | | | | | | |
| | 3" - 1 1/2" increase to each side | N | | | | | | | | | | | | | | | | | | |
| | 4" - 2" increase to each side | N | | | | | | | | | | | | | | | | | | |
| Reduce PLA / Well Sizing | 1" - 1/2" decrease to each side | N | | | | | | | | | | | | | | | | | | |
| | 2" - 1" decrease to each side | N | | | | | | | | | | | | | | | | | | |
| | 3" - 1 1/2" decrease to each side | N | | | | | | | | | | | | | | | | | | |
| | 4" - 2" decrease to each side | N | | | | | | | | | | | | | | | | | | |
| Move PLA / Well | 1" Forward | N | | | | | | | | | | | | | | | | | | |
| | 2" Forward | N | | | | | | | | | | | | | | | | | | |
| | 3" Forward | N | | | | | | | | | | | | | | | | | | |
| Base Modifications | | | | | | | | | | | | | | | | | | | | |
| Notch for Seat Rails | 1.25" from side, 1.5" from bottom | N | | | • | • | • | • | • | | | | | • | • | • | • | • | • | • |
| Notch for Back Posts | 1.25" from side, 1.5" from back | N | • | • | | | | | | • | | | | • | • | • | • | • | • | • |
| Leg Length Cut Right (left will be longer than right by specified length) | 1" | N | • | • | | | | | | | | | | | | | | | | |
| | 2" | N | • | • | | | | | | | | | | | | | | | | |
| | 3" | N | • | • | | | | | | | | | | | | | | | | |
| | 4" | N | • | • | | | | | | | | | | | | | | | | |

Note: Some limitations and restrictions apply. Please contact Customer Service for more information.

- Bullet indicates that the modification is available on the product
- Gray box indicates that the modification is not available on the product
- Green box indicates upcharge on modification

Cushions Modification Matrix (Continued)

| | | Cushions | | | | | | | | | | | | | | | | | | | |
|---|------------------------|-----------------|-------------------|------------------------|----------------|------------------|------------------|--------------------|------------------|--------------------------|------------------|------------------|-------------------|-----------------|----------------------------|-------------------|---------------------|-----------------|----------------------------|-----------------|---|
| Modifications Offered | Additional Details | Upcharge Y or N | JAY Basic Cushion | JAY Basic PRO® Cushion | JAY 60 Cushion | JAY Zip® Cushion | JAY Ion® Cushion | JAY Union® Cushion | JAY LITE Cushion | JAY Soft Combi P Cushion | JAY Care Cushion | JAY Duo® Cushion | JAY Easy™ Cushion | JAY X2® Cushion | JAY J2®/J2 Recline Cushion | JAY J2 DC Cushion | JAY J2 Plus Cushion | JAY GS® Cushion | JAY Fusion® Series Cushion | JAY J3® Cushion | |
| Leg Length Cut Left (right will be longer than left by specified length) | 1" | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 2" | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 3" | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 4" | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Contracture Cut | 1" x 45° | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Scrotal Relief Cut | | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Personalized Drop Seat | | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Angle Cut Posterior Slope (front of cushion higher than the back) | 1/2" | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 1" | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Angle Cut Anterior Slope (back of cushion higher than front by selected dimension) | 1/2" | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 1" | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 1 1/2" | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Wedge Glue | Posterior Slope | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | Anterior Slope | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Medial Support Cut | | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Femoral Relief Cut | | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Leg Trough Cut 1" Deep | Right | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | Left | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Trochanteric Relief Cut | | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Fixed Lateral Thigh Positioning Pieces | 1" Pair | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 1" Right only | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 1" Left only | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 2" Pair | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 2" Right only | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Fixed Medial Thigh Positioning Piece | 2" Left only | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 1" | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 2" | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Fixed Lateral Pelvic Positioning Pieces | 1" Pair | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 1" Right only | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 1" Left only | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 2" Pair | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 2" Right only | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Removable Lateral Thigh Positioning Pieces | 2" Left only | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 1" Pair | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 1" Right only | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Removable Medial Thigh Positioning Piece | 1" Left only | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 1" | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Removable Lateral Pelvic Positioning Pieces | 1" Pair | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 1" Right only | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| | 1" Left only | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| Visco Foam Overlay | | Y | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |
| DO NOT Install Positioning Pieces | Pieces ship separately | N | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • | • |

Cushions Modification Matrix (Continued)

| | | Cushions | | | | | | | | | | | | | | | | | | |
|-----------------------------------|---|-----------------|-------------------|------------------------|----------------|------------------|------------------|--------------------|------------------|--------------------------|------------------|------------------|-------------------|-----------------|----------------------------|-------------------|---------------------|-----------------|----------------------------|-----------------|
| Modifications Offered | Additional Details | Upcharge Y or N | JAY Basic Cushion | JAY Basic PRO® Cushion | JAY GO Cushion | JAY Zip® Cushion | JAY Ion® Cushion | JAY Union® Cushion | JAY LITE Cushion | JAY Soft Combi P Cushion | JAY Care Cushion | JAY Duo® Cushion | JAY Easy® Cushion | JAY X2® Cushion | JAY J2®/J2 Recline Cushion | JAY J2 DC Cushion | JAY J2 Plus Cushion | JAY GS® Cushion | JAY Fusion® Series Cushion | JAY J3® Cushion |
| Fluid Insert Modifications | | | | | | | | | | | | | | | | | | | | |
| Underfill | Up to 20% (in 5% increments) | N | | | | | | • | | | | | | • | • | • | • | • | • | • |
| Overfill | Up to 40% (in 5% increments) | N | | | | | | • | | | | | | • | • | • | • | • | • | • |
| Obliquity Underfill | Left or Right Side, up to 20% | N | | | | | | • | | | | | | | | | | | | |
| Obliquity Overfill | Left or Right Side, up to 40% | N | | | | | | • | | | | | | | | | | | | |
| Sacral Seal | | N | | | | | | | | | | | | | • | • | • | | | |
| Scrotal Seal | | N | | | | | | | | | | | | | • | • | • | | | |
| Cover Modifications | | | | | | | | | | | | | | | | | | | | |
| Zippered Inner Cover | Reverse Dartex® with Aquaguard® Zipper | Y | | | • | | | | • | • | • | • | | | • | • | • | • | • | • |
| | Incontinent Showercap Cover (J2 Style) | N | | | • | • | • | • | | • | • | • | | | • | • | • | • | • | • |
| | Incontinent Zipper Cover | Y | | • | | | | | | | | | | | • | | | | | |
| Personalized Outer Cover | JAY Stretch Zipper Cover (Fusion Style) | Y | | • | • | | | | | • | • | • | • | | • | | | | | |
| | X-static® Cover with Breathable Spacer Fabric | Y | | • | • | | | | | • | • | • | • | | • | | | | | |
| | 3DX® Cover with Zipper | Y | | • | | • | • | • | | • | • | • | • | | • | | | | | |
| | Dartex® Cover with AquaGuard® Zipper | Y | | • | | • | • | • | | • | • | • | • | | • | | | | | |
| | Reverse Dartex® Cover with AquaGuard® Zipper | Y | | • | | • | • | • | | • | • | • | • | | • | | | | | |
| Embroidery | Arial Font up to 32 Characters - Single color | Y | • | • | • | • | • | • | | • | • | • | • | | • | | | | | |
| | Digital Camo | N | | | | | | | | | | | | | • | • | • | • | • | • |
| | Electric Lime | N | | | | | | | | | | | | | • | • | • | • | • | • |
| | Grey | N | | | | | | | | | | | | | • | • | • | • | • | • |
| | Hunters Orange | N | | | | | | | | | | | | | • | • | • | • | • | • |
| | Navy Blue | N | | | | | | | | | | | | | • | • | • | • | • | • |
| | Pink Sorbet | N | | | | | | | | | | | | | • | • | • | • | • | • |
| | Shocking Pink | N | | | | | | | | | | | | | • | • | • | • | • | • |
| | Turquoise | N | | | | | | | | | | | | | • | • | • | • | • | • |
| | University Red | N | | | | | | | | | | | | | • | • | • | • | • | • |
| | Very Violet | N | | | | | | | | | | | | | • | • | • | • | • | • |
| | Volt Yellow | N | | | | | | | | | | | | | • | • | • | • | • | • |
| Add Front Zipper Pocket | | Y | • | • | | • | • | • | | • | • | | • | | • | • | • | • | • | • |

Note: Some limitations and restrictions apply. Please contact Customer Service for more information.



Bullet indicates that the modification is available on the product



Gray box indicates that the modification is not available on the product



Green box indicates upcharge on modification

JAY Your Way Backs Modification Matrix

| | | Backs | | | | | | | | | | |
|-------------------------------------|---|-----------------|---------------------|----------------|--------------|-----------------|-------------------|-------------|--------------|---------------|-----------------|-----------------|
| Modifications Offered | Additional Details | Uncharge Y or N | JAY Focus/Tail Back | JAY Basic Back | JAY J2™ Back | JAY J2™ DC Back | JAY J2™ Plus Back | JAY Go Back | JAY J3® Back | JAY Zip™ Back | JAY J3® HV Back | JAY J3® CF Back |
| Military Service Patch | Purple Heart | N | | | | | | • | • | | • | • |
| | Marines | N | | | | | | • | • | | • | • |
| | Army | N | | | | | | • | • | | • | • |
| | Air Force | N | | | | | | • | • | | • | • |
| | Navy | N | | | | | | • | • | | • | • |
| | Coast Guard | N | | | | | | • | • | | • | • |
| Spinal Fluid Pad | | Y | • | | • | • | • | | • | • | | • |
| Visco Foam Insert | | Y | • | | • | • | • | | • | • | | • |
| Scapular Relief Cut | | N | • | | • | • | • | | • | | • | |
| Upside Down Back Shell w/ Head rest | | N | • | | • | • | • | | • | | • | |
| Upside Down Back Cover | | N | | | | | | | • | | | |
| Zipper at the Top of the Cover | | N | | | | | | | • | | | |
| Personalized Outer Cover | Sealed Inner Cover | Y | • | • | • | • | • | • | • | | | • |
| | Incontinent Zipper Outer Cover | Y | • | • | • | • | • | • | • | | | • |
| | X-static® Cover with Breathable Spacer Fabric | Y | • | • | • | • | • | • | • | | | • |
| | 3DX® Cover with Zipper | Y | • | • | • | • | • | • | • | | | • |
| | Reverse Dartex® Cover with Aquaguard® Zipper | Y | • | • | • | • | • | • | • | | | • |
| Embroidery | Arial Font up to 32 Characters - Single color | Y | • | • | • | • | • | • | • | | | • |
| Colored Outer Cover with Zipper | Digital Camo | N | | | | | | | • | • | • | • |
| | Electric Lime | N | | | | | | | • | • | • | • |
| | Grey | N | | | | | | | • | • | • | • |
| | Hunters Orange | N | | | | | | | • | • | • | • |
| | Navy Blue | N | | | | | | | • | • | • | • |
| | Pink Sorbet | N | | | | | | | • | • | • | • |
| | Shocking Pink | N | | | | | | | • | • | • | • |
| | Turquoise | N | | | | | | | • | • | • | • |
| | University Red | N | | | | | | | • | • | • | • |
| | Very Violet | N | | | | | | | • | • | • | • |
| Painted Back Shell | All Quickie & Zippie Colors available | N | | | • | • | | | • | • | • | |
| Additional Hardware Choice | Basic Back 4-Point Locking Hardware | Y | | • | | | | | | | | |

Arm Trough and Foot Box Modification Matrix

| Arm Troughs & Foot Boxes | | | | |
|----------------------------|--------------------|-----------------|-------------|------------|
| Modifications Offered | Additional Details | Upcharge Y or N | Arm Troughs | Foot Boxes |
| | | | | |
| Add Pommel to Hand Pad | | Y | • | ■ |
| Visco Overlay | | Y | • | • |
| Add Additional Strap | | Y | • | • |
| Reduce Height of Side Wall | 1/2" | N | • | ■ |
| | 1" | N | • | ■ |
| Remove Side Walls | | N | • | ■ |
| Colored Side Bands | Digital Camo | N | • | • |
| | Electric Lime | N | • | • |
| | Grey | N | • | • |
| | Hunters Orange | N | • | • |
| | Navy Blue | N | • | • |
| | Pink Sorbet | N | • | • |
| | Shocking Pink | N | • | • |
| | Turquoise | N | • | • |
| | University Red | N | • | • |
| | Very Violet | N | • | • |
| | Volt Yellow | N | • | • |
| | Naugahyde | N | • | • |
| | Sheep Skin | N | • | • |
| Reduce Width | 3DX Microclimatic | N | • | • |
| | 1" | N | ■ | • |
| Increase Leg Trough Size | 1/2" | N | ■ | • |
| | 1" | N | ■ | • |
| Reduce Leg Trough Size | 1/2" | N | ■ | • |
| | 1" | N | ■ | • |

Note: Some limitations and restrictions apply. Please contact Customer Service for more information.



Bullet indicates that the modification is available on the product



Gray box indicates that the modification is not available on the product



Green box indicates upcharge on modification



To order JAY products with JAY Your Way modifications, please log on to your my Sunrise account and see ordering details below. If you do not have a Sunrise Medical account, please contact your local Sunrise Medical Dealer or call Customer Service at 800.333.4000

How to Order JAY Products using JAY Your Way

- Step 1:** Assess your client and select the appropriate JAY products that need to be modified.
- Step 2:** Visit www.sunrisemedical.com and log on to your My Sunrise account. To set up an account click on "Sign up for an Online Account" and follow the prompts.
- Step 3:** Begin a "New Order" or "New Quote".
- Step 4:** Enter an "M", followed by the item number of the JAY product you would like to modify. For example, "MJFUSION1818", for a modified Fusion cushion.
- Step 5:** Select from the numerous drop-down menus for the modifications you require. If you do not see a modification you require in this catalog or in the drop-down menus, please contact Customer Service. They will advise if that modification can be made for that product.